



	BUTTONS	PUMP ACTIVATION	SCREEN
INSERTING BATTERY PHASE		<ul style="list-style-type: none"> The screen will show all symbols. Piston self-adjust. Pump switches OFF. 	<p>88888</p> <p>----</p> <p>OFF</p>

	BUTTONS	REVERSING OF THE PUMP PISTON NUMBER OF INFUSIONS DELIVERED	SCREEN
PUMP OFF	depress simultaneously	Reversing of the pump piston to the start position.	End ---- OFF
	depressed for 8 seconds	Reading the number of infusions delivered.	0015
	depress simultaneously	Reset the number of bolus delivered (L0).	
		Confirm the reset of bolus delivered.	00

KEY-PAD LOCK-OUT			
PUMP OFF		Press for 7 seconds.	0015
	press without releasing button	Key-pad lock level selection.	L0 L1

PUMP ON			
		Pump switches ON.	

PCA 50 CRONO

	BUTTONS	SETTINGS	SCREEN
PUMP OFF		Operations feasible only with key-pad unlocked.	L0
	1 st depressing	<ul style="list-style-type: none"> Setting flow (from 0.05 to 35 ml/h with the following increments:- from 0.05 to 10 with 0.02 ml increments; - from 10 to 35 with 0.05 ml increments). 	F5.48
	2 nd depressing	<ul style="list-style-type: none"> Setting bolus dose (from 0.10 to 9.98 ml with 0.02 ml increments). 	20.10
	3 rd depressing	<ul style="list-style-type: none"> Setting bolus time limitations (- from 5' to 1 h with increments of 5' - from 1 h to 24 h with increments of 15'). 	120'
	4 th depressing	<ul style="list-style-type: none"> Setting boluses number limitations (0-12 per h). <p>Bolus lock-out possibilities: combination of bolus time and bolus number limitations. The computer always chooses the programming that results in the smallest bolus volume.</p>	n. 10
	5 th depressing	<ul style="list-style-type: none"> Setting partial volume (from 20 to 50 ml with increments of 1 ml - possible only at the begin of a new infusion). 	45cc
		<ul style="list-style-type: none"> Decrease/Increase above parameters. 	

	BUTTONS	PRIME	SCREEN
PRIME CONDITION		Priming possible: only at start of a new infusion or start of a new partial infusion.	Pr
		Access to PRIME phase.	Pr
		PRIME (each depressing administers 0.5 ml - max 1.5 ml).	P050
		PRIME phase finish, infusion start.	F5.48
	and depress simultaneously	PRIME interruption (if in execution).	Pr
	and depress simultaneously	Device switch OFF (if PRIME carried out).	OFF
		In case there is a need to prime one or more times during an already started infusion, the clinician bolus can be used	

	BUTTONS	INFUSIONS	SCREEN
PUMP ON		• Bolus dose delivery.	
	and depress simultaneously	• Bolus interruption (if in execution).	

	BUTTONS	SCREEN REVIEW OPTIONS	SCREEN
ON	1 st depressing	• Administered flow rate in ml since start of infusion.	
	2 nd depressing	• Administered patient bolus in ml since start of infusion.	
	3 rd depressing	• Administered number of bolus.	
	4 th depressing	• Administered clinician bolus in ml since start of infusion.	
	5 th depressing	• Total administered volume in ml since start of infusion.	

	BUTTONS	SCREEN REVIEW OPTIONS	SCREEN
ON	1 st depressing	• Programmed flow rate.	
	2 nd depressing	• Programmed bolus dose.	
	3 rd depressing	• Programmed bolus time limit.	
	4 th depressing	• Programmed bolus number limit.	
	5 th depressing	• Programmed partial volume.	
	6 th depressing	• Programmed clinician bolus dose.	

PCASO

AUG 01/USA00 PCA50 07/05

	BUTTONS	CLINICIAN BOLUS FUNCTION	SCREEN
PUMP OFF	Press for 7 seconds.		
	press without releasing button	• Key-pad lock level selection.	
		• Select clinician bolus level.	
		• After the screen will show the latest administered clinician bolus dose.	
	/	• Decrease/Increase clinician bolus (from 0.10 to 9.98 ml with 0.02 ml increments).	
		• Clinician bolus delivery.	
	and depress simultaneously	• Clinician bolus function interruption.	
		After the administration of one clinician bolus, the lock level L 2c switches automatically back to the lock level programmed when the clinician bolus procedure started i. e. either L 0 or L 1 and no more clinician bolus can be delivered without repeating the clinician bolus procedure from the beginning.	

	BUTTONS	PUMP OFF	SCREEN
PUMP ON	and depress simultaneously	• Pump switches OFF.	

	BUTTONS	END OF INFUSION	SCREEN
END INFUSION	and depress simultaneously	• End of infusion. • Piston automatic reversing. • Automatic switch OFF.	

Distributed by: Intra Pump Infusion Systems 920 Minters Chapel Rd, Suite 200, Grapevine, TX 76051
Toll Free: 1-866-211-7867(pump) Fax: 630-845-2768
www.intrapump.com



Man.: Canè S.r.l. Medical Technology Rivoli (TO) - Italy