


# CRONO *five*

## PHYSICIAN SHORT INSTRUCTION

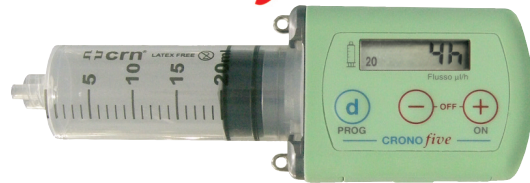

	BUTTONS	PUMP ACTIVATION	SCREEN
INSERTING BATTERY PHASE	+ / -	<ul style="list-style-type: none"> <li>The screen will show all symbols.</li> </ul>	
		<ul style="list-style-type: none"> <li>Syringe size selection.</li> <li>Pusher self-adjust.</li> <li>Pump switches OFF.</li> </ul>	
PUMP OFF	REVERSING OF THE PUMP PISTON NUMBER OF INFUSIONS DELIVERED		SCREEN
	depress simultaneously  depressed for 7 seconds	<ul style="list-style-type: none"> <li>Reversing of the pump piston to the start position.</li> </ul>	
		<ul style="list-style-type: none"> <li>Reading the number of infusions delivered.</li> </ul>	
	depress simultaneously  	<ul style="list-style-type: none"> <li>Reset the number of boluses delivered (L0).</li> </ul>	
		<ul style="list-style-type: none"> <li>Confirm the reset of boluses delivered.</li> </ul>	
KEY-PAD LOCK-OUT			
PUMP OFF	 press without releasing button 	<ul style="list-style-type: none"> <li>Press for 7 seconds.</li> </ul>	
		<ul style="list-style-type: none"> <li>Key-pad lock level selection.</li> </ul>	
		<ul style="list-style-type: none"> <li>Selection of the lock level.</li> </ul>	
PUMP ON			
	+	<ul style="list-style-type: none"> <li>Pump switches ON.</li> </ul>	

CRONO *five*

	BUTTONS	SETTINGS	SCREEN
PUMP OFF		Operations feasible only with key-pad unlocked.	
	1 <sup>st</sup> depressing	<ul style="list-style-type: none"> <li>Setting flow rate</li> </ul>	
	2 <sup>nd</sup> depressing	<ul style="list-style-type: none"> <li>Setting bolus dose</li> </ul>	
	3 <sup>rd</sup> depressing	<ul style="list-style-type: none"> <li>Setting bolus time limitations</li> </ul>	
	4 <sup>th</sup> depressing	<ul style="list-style-type: none"> <li>Setting bolus number limitations</li> </ul>	
	5 <sup>th</sup> depressing  + / -	<ul style="list-style-type: none"> <li>Setting partial volume</li> <li>Decrease/Increase above parameters.</li> </ul>	
PRIME CONDITION	PRIME		SCREEN
		<ul style="list-style-type: none"> <li>Access to PRIME phase.</li> </ul>	
		<ul style="list-style-type: none"> <li>PRIME (each depressing administers 0.5 ml - max 1.5 ml).</li> </ul>	
		<ul style="list-style-type: none"> <li>PRIME phase finish, infusion start.</li> </ul>	
	depress simultaneously  depress simultaneously	<ul style="list-style-type: none"> <li>PRIME interruption (if in execution).</li> <li>Device switch OFF (if PRIME carried out).</li> </ul>	
In case there is a need to prime one or more times during an already started infusion, the clinician bolus can be used			

	BUTTONS	INFUSIONS	SCREEN
PUMP ON		• Bolus dose delivery.	
	depress simultaneously	• Bolus interruption (if in execution).	

	BUTTONS	SCREEN REVIEW OPTIONS	SCREEN
PUMP ON	1 <sup>st</sup> depressing	• Administered flow rate in ml since start of infusion.	
	2 <sup>nd</sup> depressing	• Administered patient bolus in ml since start of infusion.	
	3 <sup>rd</sup> depressing	• Administered number of boluses.	
	4 <sup>th</sup> depressing	• Administered clinician bolus in ml since start of infusion (available only to the physician).	
	5 <sup>th</sup> depressing	• Total administered volume in ml since start of infusion.	

	BUTTONS	SCREEN REVIEW OPTIONS	SCREEN
PUMP ON	1 <sup>st</sup> depressing	• Programmed flow rate.	
	2 <sup>nd</sup> depressing	• Programmed bolus dose.	
	3 <sup>rd</sup> depressing	• Programmed interval time between boluses.	
	4 <sup>th</sup> depressing	• Programmed number of boluses per hour.	
	5 <sup>th</sup> depressing	• Programmed partial volume.	
	6 <sup>th</sup> depressing	• Programmed clinician bolus dose (available only to the physician).	

Crono five

AUG 01/USA/04 Crono FIVE 07/15

	BUTTONS	PUMP OFF	SCREEN
PUMP ON	depress simultaneously	• Pump switches OFF.	

	BUTTONS	END OF INFUSION	SCREEN
END OF INFUSION	depress simultaneously	<ul style="list-style-type: none"> <li>• End of infusion.</li> <li>• Piston automatic reversing.</li> <li>• Automatic switch OFF.</li> </ul>	

**INTRA PUMP INFUSION SYSTEMS**  
 920 Minters Chapel Road, Suite 600 Grapevine, Texas 76051  
 Tel: 866-211-7867 Fax: 630-845-2768 Email: info@intrapump.com  
**CANÈ S.p.A. MEDICAL TECHNOLOGY**  
 Via Cuornè 42/a, 10098 Rivoli (TORINO) Italy  
 Tel: 0039-011-9574872 Fax:0039-011-9598880 Email: mailbox@canespa.it