



neria™ infusion sets

The neria[™] infusion set is a steel needle infusion set designed with ease of use and discreet wear in mind. The neria[™] infusion set is inserted at a 90 degree angle. The simple and quick insertion method supports patient self management and is a suitable choice when using continuous pump therapy.

Product Specifications

Cannula: Stainless Steel

nless Steel Adhesive:

Built-in, skin friendly¹ non-woven polyester, acrylic adhesive

Disconnection: No

Insertion angle:

Gauge	Needle	Tubing	Priming volumes
size	lengths	lengths	
G27, G29	6, 8 and 10 mm	60, 80 and 110 cm	60 cm ~ 0.10 ml 80 cm ~ 0.12 ml 110 cm ~ 0.15 ml

90 degrees (straight in)

The neria[™] infusion sets are sold in boxes containing 10 sets.


Reference: ¹ Biological Safety Testing, October 2011, Data on File, Unomedical a/s.









neria™ infusion sets user steps

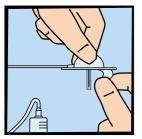


Prime the tubing according to pump manufacturer's instructions for use and remove the adhesive backing paper.

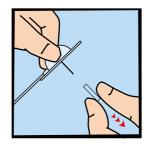

2. Gently twist and pull to remove the needle guard.

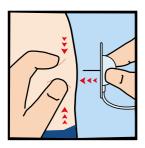

3. Pinch the skin with one hand and insert neria™ infusion set vertically.

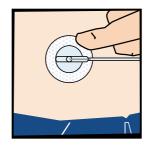

4. Press the adhesive to secure neria™ infusion set to the skin. Check the infusion site frequently.